Beautifully Unique

Products Care Instructions

Thank you for your purchase with Beautifully Unique, LLC. With a little care, they will provide lasting comfort. Please follow the care instructions below.

Breast Form Care Instruction:

- Clean your breast form *daily* in lukewarm water with mild soap and dry off with a soft towel. Chlorinated or salt water will not harm your breast form.
- Avoid stretching, twisting, rubbing or using excessive force when handling your breast form,.
- Store your breast form nipple down in the travel case or box provided.
- Avoid bringing your breast form in direct contact with perfume, hot surfaces, and hot water or abrasive cleaners.
- Avoid sharp, pointed objects near your breast form. Pins, fingernails, pet claws, scissors, etc. can all cause permanent damage.

Bra Care Instructions

- Hand wash or machine wash on gentle cycle your bra in warm water with a mild detergent.
- Fasten all the fasteners and recommended to put in a laundry bag.
- **Do not use** fabric softener or bleach.
- Line dry **do not** machine dry.
- Hang by both shoulder straps on a non-wire hanger.
- Form breast cups by hand.

Beautifully Unique, LLC

Women's Specialty Boutique

2004 Inverway Ct. Kalamazoo, MI 49009

(269) 312-8733

FAX (269) 312-8161

http://beautifullyunique.us